



MICROCRYSTALLINE BLACKHEAD INSTRUMENT CC-9100

Note: Please read the instruction carefully before use

Product introduction and working principle

Microcrystalline blackhead tester is a collection of skin grinding, tightening, pore cleaning, acne removal, blackhead/acne absorption and oil fat. Multifunction beauty equipment such as using natural mineral microcrystalline drilling particles with physical vacuum suction device to remove the other aging cuticle and dirt in pores, make pore becomes fine, color white uniform and smooth, and it is non-invasive and no stimulative technology, by the suction of diamond on the rod thickness to control the depth of exfoliating when degrees. At the same time, 6 probes of different shapes have 6 different functions, which can be used for skin grinding, pore cleaning and other functions.

Efficacy Characteristics

Fine and smooth skin: fine and skin care can gently remove aged Cutin and make skin tender and smooth.
Tighten V face: Vacuum absorb bullet, lift, and tighten skin, improved double chin, make V face. Blackhead suction: Can strongly absorb the black head if the skin, oil, and dirt, clean the skin.
Fade fine lines: lift and massage the skin to increase vitality and fade fine lines.
Large screen display: with a large screen display, more clearly display your current mode.

Precautions

1. Dry the water or other liquid on your hands before use to avoid water or other liquids from the body. The vent hole on the back enters the body and damages the body.
2. Do not use scrub or other keratin products before or after use.
3. Microdermabrasion with micro-crystal head, dry skin once a week, oily combination skin It can be used twice a week, and can be used for no more than 5 minutes each time, depending on the skin condition. And set.
4. Please do not press the tip on the skin, because the instrument will produce a strong pulling force.
5. Please do not continue to use the same part, may cause skin red purple, if the is occurs Please stop using the situation.
6. After the end of the nursing, the skin will have some irritation or redness due to the accelerated circulation of the skin. State, this is a normal phenomenon, it will automatically disappear after about 30 minutes of use. Please do a good job Insulation work.
7. This product is not suitable for physical, sensory, or mental disability or lack of experience and knowledge. Use, including not allowing children to use, if the use must have a guardian to supervise from the side, and the use of this product is guided and can withstand responsibility.
8. This product is not suitable for cosmetic surgery, rhinoplasty and other parts of the skin cosmetic surgery, pregnant women, heart disease patients are also not recommended.

Six alternative work heads

	Oval hole beauty tip It is mainly used to remove file lines. It needs to be quickly pulled up and repeatedly after sucking the skin, which can promote skin blood circulation and increase skin elasticity
	Micro crystalline beauty tips Fine crystals gently remove dead skin cells and leave skin smooth and supple.
	Mini small round hole beauty head Small suction for sensitive skin or blackheads.
	Small round hole beauty tip Small suction for blackheads. For example, if the skin is thin and tender, sensitive muscles and not suitable for too strong suction, use a small tip.
	Medium round hole beauty tip It has a large suction force and id used for sucking blackheads. If the acne is too much, do not use it in one part for a long time. Use the lifting methods from bottom to bottom and moves slowly from the chin.
	Large round hole beauty tip The suction is large, it is used to suck the blackhead and tighten the V-face. When sucking the blackhead, you need to slowly move on the skin backwards. Do not use it in one part for a long time. When shaping the V face, use the pull-up technique from bottom to top to move slowly from the chin.

Steps for usage

- Use a hot towel with solution to open the pores before use.
1. Press the suction head into the machine.
 2. Press and hold the switch button for 2-3 seconds to turn on (tap to turn off), then short press the "M" button to select suction Force strength h, a total of 3 levels of strength, the buttons are in order of "default is mid-range (neutral)", and then press the high Gear (oily), then press the low gear (dry), the size of the force can be adjusted with the size of the tip hole The suction force can be tested on the skin such as the arm to choose the appropriate strength.

3. The suction head should move slowly on the skin. Do not stay in the same area for too long. According to oneself Choose oily, neutral, and dry skin.

Facial dermabrasion using three methods

1. Cleanse the face and dry it.
2. Put on a microcrystalline beauty head or a large round hole beauty head.
3. Starting from the squat, moving the beauty head from the bottom up.
4. Place the blackhead on the cheek and move from inside to outside.
5. Place the blackhead instrument in the forehead area and move from the middle to the sides.
6. Place the blackhead in the T zone and move from bottom to top.

How to clean pores and dirt

1. Replace the large round hole beauty head
2. Align the probe hole with blackheads and acne for care. Adjust the machines position as needed.

Manipulation: When sucking blackheads, you can move slowly in the part with blackheads, repeat multiple times, do not go back and forth Move, it should be moved according to the skin texture method. For acne and acne, please pull it out immediately after sucking.

Reduce the use offline lines

1. Change the oval hole beauty.
 2. Holding the machine in the corner of the eye, nose, lips, and other part s for care, can adjust the machine according to needs Gear posit ion.
- Manipulation: You can quickly pull up and pull on the skin area you need, and then lift Pull, so repeated operations, acting on the bottom layer of the skin, promoting firmness and increasing skin elasticity, Prevent wrinkles.

Skin property resolution method

1. Dry Skin
Dry skin is generally less oil and moisture secreted from the face, dry skin face The pores on the skin are not obvious, the skin is always in a relatively dry and refreshing state, but When the season is changed or winter, the skin will look very dry and even peel. Currently, the skin has the longest wrinkles and is not very smoot h and thick. Rough, this is called dry skin.

2. Oily Skin
Oily skin means that there is too much oil an t he faces, it is easy to have large pores, and the face looks good. It will brighten and it will accumulate oil in the pores, but this kind of skin will not peel off, nor will it, It is easy to wrinkle, but oily skin is easy to get acne, and people who are too fat are very easy It is oily skin.

3. Neutral skin
Neutral skin is a combination of dry skin and oily skin. It is also considered by people to be the most. For the healthiest skin, it is not easy to secrete oil or dry. Dry; not easy to wrinkle, not easy to peel, and the skin is elastic and shiny, is the most. The ideal skin is gone.

Features



(1) Functional beauty head (replaceable)	(3) Power Button
(2) Mode suction adjustment button	(4) Display

Specifications

Product Name	Micro-crystalline Blackhead	Rated Voltage	5V-
Product Number	CC-9100	Rated Current	0.5A
Product Size	170x42x36mm	Rated Power	2.5W
Product Weight	118g	Battery Capacity	500mAh
Product Material	ABS+PC	Gear Adjustment	Third Gear
Origin	China	Suction	-58KPa

Product Warranty Card

Product Name: Micro-crystalline Blackhead	Model:
Store:	
Name:	
Address:	
Telephone:	
Fault Description:	



CERTIFICATE



DATE OF PRODUCTION:
EXAMINATION CLERK:
IMPLEMENTATION CRITERIA: GB4706.I-2005, GB4706.I-2005

